

# OUR BEST-IN-CLASS C-MAC® POCKET MONITOR JUST GOT BETTER




## Announcing a New Software Upgrade

The new software upgrade (747 v 104) offers new standby mode notification symbols. The two-minute and one-minute indicators provide a heads-up that the device will soon enter standby mode.

- Any C-MAC® Pocket Monitor with serial number XR3182 and above will have the new software installed
- Units that are sent in for repair will come back with the new software installed

# OUR BEST-IN-CLASS C-MAC® POCKET MONITOR JUST GOT BETTER

## Announcing a New Software Upgrade

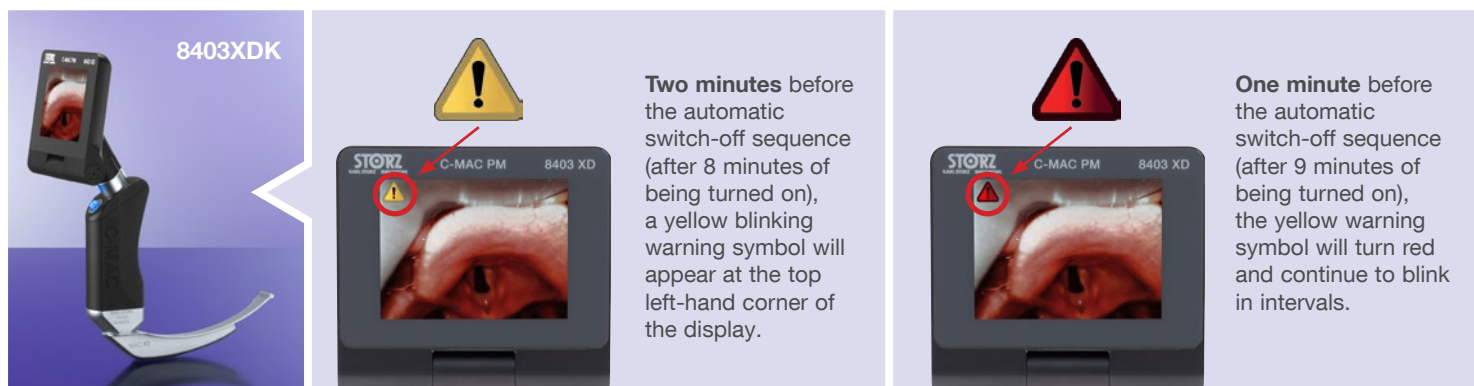

The 10-minute power save feature can be extended by the use using the following methods:

- Press the Blue Button on the connected device (Blade, S-Imager or Video Stylet) — note that this takes an image
- Close and reopen the OTI® display
- Disconnect and reconnect the Blade, S-Imager or Video Stylet

The 10-minute cycle is deactivated while recording videos. It is restarted after the video recording is completed.

It is recommended to check the suitability of the product for the intended procedure prior to use.

KARL STORZ SE & Co. KG  
Dr.-Karl-Storz-Straße 34, 78532 Tuttlingen/Germany  
Postbox 230, 78503 Tuttlingen/Germany  
Phone: +49 (0)7461 708-0  
Fax: +49 (0)7461 708-105  
E-Mail: [info@karlstorz.com](mailto:info@karlstorz.com)  
[www.karlstorz.com](http://www.karlstorz.com)

**STORZ**  
KARL STORZ — ENDOSKOPE  
THE DIAMOND STANDARD

KARL STORZ Endoscopy-America, Inc.  
2151 East Grand Avenue  
El Segundo, CA 90245-5017, USA  
Phone: +1 424 218-8100  
Phone toll free: 800 421-0837 (US only)  
Fax: +1 424 218-8525  
Fax toll free: 800 321-1304 (US only)  
E-Mail: [communications@karlstorz.com](mailto:communications@karlstorz.com)