

SMARTER DESIGN

Easily Integrate Into Your Operating Room

UH 400
Electrosurgical Unit



SET UP IS SIMPLE

Select, Connect, Start


STORZ
KARL STORZ—ENDOSKOPE

Select

To Prepare for Surgery, Simply Power on the UH 400 and **Select** the Correct Settings

Select Program from Favorites 


Favorites Menu

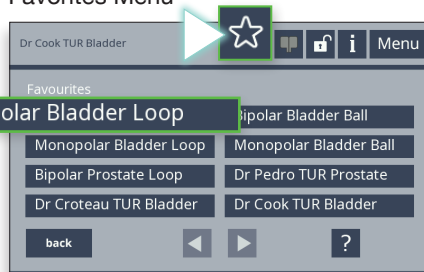

OR

Select from Program List

Program List

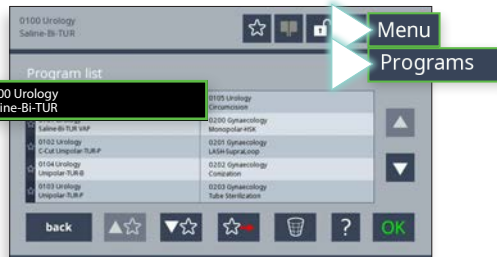

Connect High Frequency Cable

Monopolar on the Left

Bipolar on the Right



	Effect	Pedal	Mode	max. Watt
off	2 500 Vp			
	3 190 Vp			

Check Pedal

Orange circle  means Pedal is on

Assemble the Electrode and Resectoscope



Listen for a “click”. This confirms the electrode is locked.
Connect the HF cable.

Setup is now complete!

READY TO START



Contact your KARL STORZ representative, or call
800-421-0837 to experience Smarter Energy, delivered
exclusively with the UH 400.

STORZ
KARL STORZ—ENDOSKOPE