

THE SUPRALOOP – FOR LAPAROSCOPIC SUPRACERVICAL HYSTERECTOMY (LASH)



Click or scan the QR code to access the study “Laparoscopic supracervical hysterectomy with the SupraLoop”.



Faster surgeries with your choice of instrument

The SUPRALOOP was specially designed to support the operating surgeon when detaching the uterine corpus during LASH procedures.

- Shorter operating time thanks to quick removal of the uterine corpus when compared to detachment with a unipolar needle*
- Loop available in two sizes to match anatomical conditions
- Reusable handle and outer sheath

* Brucker S, Rothmund R, Krämer B, Neis F, Schönfisch B, Zubke W, Taran FA, Wallwiener M. Cervical Detachment Using Monopolar SupraLoop Electrode versus Monopolar Needle in Laparoscopic Supracervical Hysterectomy (LSH): An Interventional, Comparative Cohort Study. Geburtshilfe Frauenheilkd. 2013 Nov;73(11):1121-1127. doi: 10.1055/s-0033-1350975. PMID: 24771898; PMCID: PMC3862041.

THE SUPRALOOP – FOR LAPAROSCOPIC SUPRACERVICAL HYSTERECTOMY (LASH)

Faster surgeries with your choice of instrument



The SUPRALOOP is inserted into the abdominal cavity via a 6 mm trocar. The loop is then placed around the uterus and, after carefully checking for correct positioning, the loop is retracted...



...and the uterus is detached by applying unipolar or bipolar HF current.

No matter whether you prefer a unipolar or bipolar current for laparoscopic supracervical hysterectomy - the choice is yours:

Unipolar SUPRALOOP	
26183M	BRUCKER/MESSROGHLI SUPRALOOP Set , unipolar, size 5 mm, length 30 cm, consisting of: 26183MA SUPRALOOP Handle 26183MB SUPRALOOP Outer Sheath 26183MC SUPRALOOP Spare Loop
Accessories:	
26183MC	SUPRALOOP Spare Loop , unipolar, size 120 x 85 mm, unsterile, for single use
26183MD	SUPRALOOP Spare Loop , unipolar, size 200 x 150 mm, unsterile, for single use
Bipolar SUPRALOOP	
26183N	SUPRALOOP Set , bipolar, size 5 mm, length 30 cm, consisting of: 26183NA SUPRALOOP Handle 26183MB SUPRALOOP Outer Sheath 26183NC SUPRALOOP Spare Loop
Accessories:	
26183NC	SUPRALOOP Spare Loop , bipolar, size 120 x 85 mm, unsterile, for single use
26183ND	SUPRALOOP Spare Loop , bipolar, size 200 x 150 mm, unsterile, for single use
HF Generator	
UH400	AUTOCON® III 400 High-End , without CE certification, power supply 220-240 VAC, 50/60 Hz
UH400E	AUTOCON® III 400 High-End , with CE certification, power supply 220-240 VAC, 50/60 Hz

You can contact us directly via email at gyn@karlstorz.com

It is recommended to check the suitability of the product for the intended procedure prior to use.
Please note that the described products in this medium may not be available yet in all countries due to different regulatory requirements.

KARL STORZ SE & Co. KG
Dr.-Karl-Storz-Straße 34, 78532 Tuttlingen/Germany
Postbox 230, 78503 Tuttlingen/Germany
Phone: +49 7461 708-0
Fax: +49 7461 708-105
E-Mail: info@karlstorz.com
www.karlstorz.com

STORZ
KARL STORZ – ENDOSKOPE

FOLLOW KARL STORZ



www.karlstorz.com

